drive™

www.drivemedical.com

Rev.3.04.15.14

plastic bathtub safety rail

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item # 12039 & 12044

EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

drive™

installation instructions

1. Clean Bathtub thoroughly and wipe dry before applying rail. Contact surfaces must be dry and free of soapy residue to assure secure installation.
2. Deploy knob (A) and open until rail is wide enough to slip on side of bathtub.
3. Place Safety Rail on bathtub with knob (A) facing outside of tub and tighten until rail feels tight and secure.
4. To test installation of rail, pull forward, back and side to side. If rail slips or moves, make sure surface is free of soapy film and is dry, then replace at desired location and retighten.

Safety rail often requires retightening from time to time. Check periodically before use and retighten as needed.

CAUTION:

Please use extreme care when installing on fiberglass tubs. **DO NOT OVERTIGHTEN** as the clamp may crack the fiberglass.

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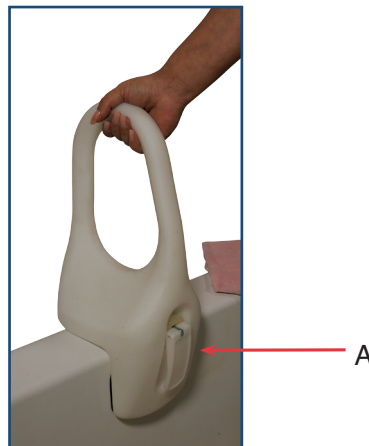
Article # 12044

lifetime limited warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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