driveTM

www.drivemedical.com

Rollator

Andador

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driveTM

EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

Assembly Instructions

Do not attempt to adjust or operate Rollator without carefully reading all instructions in this manual.

1. First remove all contents from carton: backrest, handle with tube, Rollator frame and Basket.
2. Open frame and slide padded backrest into frame brackets making sure snap bottoms protrude through holes.
3. Loosen tightening knob on handle receptacle and insert handle tube into frame. The brake cable should be positioned on outside of frame. Set handle to desired height. Tighten knob by turning clockwise until handle is securely in place. Make sure both handles are set to the same height before use.

Safety Precautions

1. Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
3. Test the loop locks before operating the Rollator. When wheels are locked (see illustrations for locking and releasing locks below), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should turn freely.
4. Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tires when significantly worn or damaged.

Important Safety Notice

Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

Loop Lock Operation

Always engage and disengage wheel locks on both sides simultaneously.

1. To Slow Down or Come to a Temporary Stop – Pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.
2. To Temporarily Lock Wheels in Position – Press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.
3. To Release Wheel Locks – Push up on both loop locks with palms of hand to original position.

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Lifetime Limited Warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2012 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2012 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2012 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.