www.drivemedical.com

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www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

installation instructions

1. Remove product from carton and read directions carefully before installing raised toilet seat.
2. Remove toilet seat and lid from toilet.
3. Turn both locking clamps counter clockwise until they can no longer turn. This will open the locking devices.
4. Place raised toilet seat on bowl and position in a secure fashion. Be sure the two locking brackets are on both sides of the bowl wall and that the seat is flush to the bowl.
5. Turn the locking brackets clockwise until tight. Hand tighten only. Do not use tools as threads on adjustment bracket may strip.
6. Check for secure fit routinely before using product. User's weight should be centered over toilet seat.

removal instructions

1. Rotate knobs counter clockwise until seat is loose.
2. Lift front of raised toilet and pull forward to move.

cleaning instructions

1. Use a non-abrasive detergent or household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may occur.

CAUTION

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that adjustment knobs on sides of seat are tight.
3. This product is designed for use with most common types of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.