

# Installation Instructions for Pivot Assist Platform

Item# 5601—With Pivot Disc Included

Item# 5602—Without Pivot Disc

1. When unpacking be careful not to remove the anti-slip material on the bottom of the platform.



2. Bolt the two handle rails together using the supplied hex head bolt. The circle marks should face each other when bolting together.





3. Insert carriage bolts into platform base from the underside. Align the handle bolt holes with the bolts in the platform base then secure the Five Star knobs and tighten securely. Re-check tightness occasionally after usage. **(Note: The main picture to the left does not show the Five Star Knobs.)**



If you ordered Item 5602 disregard step 4

## Directions for usage:

Position Platform as close to the bed as possible. Both wheelchair front wheels are to be positioned onto the platform as shown. Align the Pivot platform with a point in the bed where you wish to sit, so once the transfer is complete you are in a good position to lie down. Again roll both front wheels onto the platform as shown with your feet under you and resting on the Pivot Disk. Lock wheels before starting the transfer. Slide forward in the wheelchair seat. **Grasp handles, Stand, Pivot, and complete the transfer.** You might find it easier to stand if you place a hand on the support rail while keeping your other hand on the arm of the wheelchair. Try both methods.

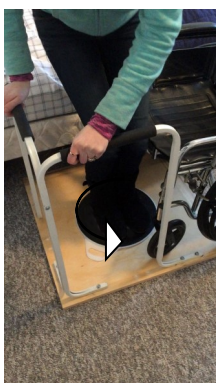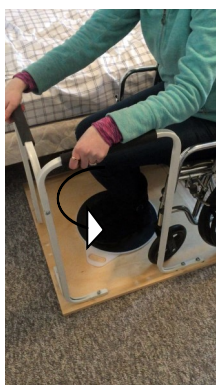

4. Place Pivot Disc into routed insert on the plywood platform. Once positioned the Pivot Disc should left in place. If for any reason the Disc does not stay in position. DISCONTINUE USE IMMEDIATELY.

Contact Customer Service 1-800-854-4687

### Contents \*5601 w/ disc \*5602 w/o disc

Support Base  
2-Handle Assembly's  
1-Hex Head Bolt / 1 Washer & 1 Nut  
6-Carriage Bolts / 6 Knobs

If bolts spins when tightening, from the underside of the platform tap carriage bolt with a hammer to set bolt into wood. This should prevent it from spinning.