


Pivot Assist Platform

Item# 5601



Mobility Transfer Systems Inc.

1. When unpacking be careful not to remove the anti-slip feet on the bottom of the platform.

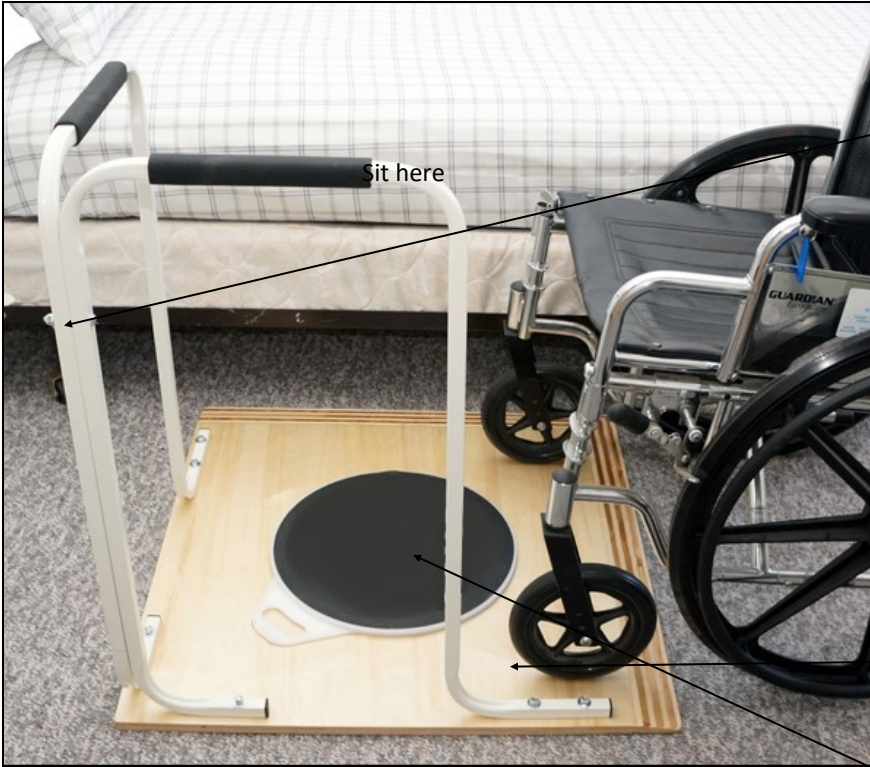

2. Bolt the two handle rails together using the supplied hex head bolt preinstalled in one of the handle legs. The red circle marks should face each other when bolting together.





3. Insert carriage bolts into platform base from the underside. Align the handle base bolt holes with the bolts in the platform base. Place the supplied washer first and then apply the nut. Tighten bolts securely. Re-check all bolt tightness after usage.



Directions for usage:

Position Platform as close to the bed as possible while still allowing both wheels to be positioned onto the platform as shown. Align the Pivot Disc with a point in the bed where you wish to sit, so once the transfer is complete you are in a good position to lie down. Roll both wheels onto the platform as shown. Lock wheels before starting the transfer. Place feet on the Pivot Disc. Slide forward in the wheelchair seat. Grasp handles, stand, pivot, and complete the transfer. You might find it easier to stand if you place your left hand on the support rail while keeping you right on the arm of the wheelchair.

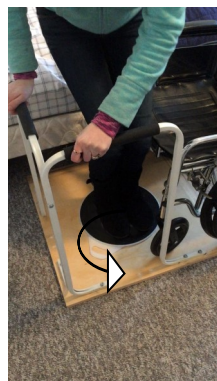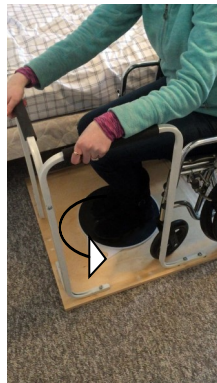

4. Place Pivot Disc into routed insert on the plywood platform. Once positioned the Pivot Disc should remain in place with sliding out. If for any reason the Disc does not stay in position. DISCONTINUE USE IMMEDIATELY.

Contact Customer Service 1-800-854-4687

Contents

- Support Base
- 2-Handle Assembly's
- 1-Hex Head Bolt / Washer & Nut
- 6-Carriage Bolts / Washers & Nuts

