

# SafetySure<sup>®</sup> Mary's Aide<sup>™</sup> Instructions

## Follow these simple instructions for safer transfers:

1. After the patient is dressed, place the Mary's Aide under them with the larger, contoured panel in the back. The back panel has the product label.
2. Bring the front panel up between the legs.
3. Attach the side belts together and secure with the Velcro<sup>®</sup> attachments.
4. Stand in front of the patient and grasp the Velcro side belts.
5. Have the patient lean forward and place their arms around your waist and begin the lift.
6. After the transfer has been made and the patient is comfortably seated, unfasten the Velcro handles and fold the Mary's Aide back and out of sight until it is needed for the next transfer.

**Laundry Tips:** Washing in cool water and air drying will extend the life of the Mary's Aide.  
To avoid lint collection in the Velcro, close the handles before washing.