Escali[®]
Measuring What's Next[™]

Model: B180 R/S
User's Guide

Precision Body Weight Scale


WWW.ESCALI.COM

Thank you for choosing an Escali Scale. This precision state-of-the-art measuring instrument is most accurate when weighing on a flat hard surface. As the electronic sensors are sensitive, avoid dropping or jarring the scale and store where it will be protected from impact.

Operating instructions

1. A replaceable lithium battery is included. Please pull the insulation tape out from below the battery before first use.
2. Select your desired weighing unit with the switch on the bottom of the scale.
3. Place the scale on a solid and flat surface. Turn on the scale by tapping on the platform with your foot; wait a few seconds for "0.0" to appear before standing on the scale.
4. Gently step onto the scale, place your feet evenly spaced, and avoid moving while the scale measures your weight.
5. After your weight is displayed, the scale will turn off automatically after approximately 6 seconds.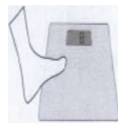

Over Load Indicator

When the scale is over loaded (over 400lb / 180 kg), "Err" will appear on the screen.

Additional Information

1. Make sure your scale is at room temperature, and on a stable horizontal surface free of vibration.
2. This scale is not waterproof. Do not expose to moisture.
3. As this scale is a precision instrument, it must be handled with extreme care. Avoid rough treatment, shaking and vibration.
4. Do not disassemble this scale or the warranty will be void.
5. The display value may be affected by electromagnetic disturbances such as operating near a radio. If this occurs, move the device that is causing the disturbance and turn the scale off and restart.

Product Guarantee

Your Escali scale is guaranteed to be free of defects in materials and workmanship for a lifetime. For full details of this guarantee please visit our web site at:

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