

SAFETYSURE® BED CRADLE


Assembly Instructions

The SafetySure Bed Cradle consist of four pieces.


Two Vertical Bars.

Two Horizontal Bars.

(figures 1 & 2)

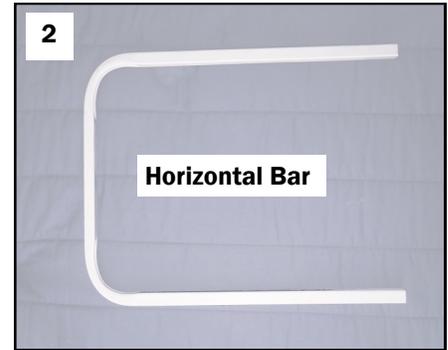

To assemble: the Bed Cradle, insert a leg of the vertical bar into one leg of the horizontal bar (see figure 3). Repeat the procedure with the other vertical bar. Press the vertical bar in as far as possible in order to lock the legs in place. Complete the assembly by attaching the other horizontal bar. Slide the complete unit between the box spring and mattress at the foot of the bed (figure 4). Make the bed by placing the top sheet and bed cover over the Bed Cradle (figure 5).

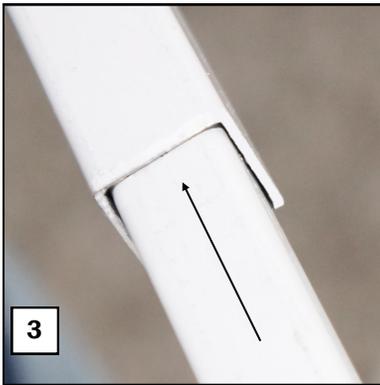

Note: If the legs do not lock with the unit fully assembled, rest horizontal bars on the floor (Image 6). Hold the vertical bars and push down hard, lift and tap the Cradle on the floor a several times pushing downward.

