drive™

[www.drivemedical.com](http://www.drivemedical.com)

Rev1.05.09.14

deluxe hand-held  
shower massager

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[www.drivemedical.com](http://www.drivemedical.com)

## installation instructions

1. Remove the existing shower head leaving shower arm.
2. Screw Anti-Siphon Device (A) into existing shower arm.
3. Place rubber gasket (included) into Rotator Joint (B) and screw Rotator Joint into the other end of the Anti-Siphon Device.
4. Locate the end of the hose that has a bolt on the end and screw it onto the Rotator Joint (B).
5. Screw the other end of hose into the new shower head (C).
6. Hang shower head on bracket end of Anti-Siphon Device (A) when not in use.

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
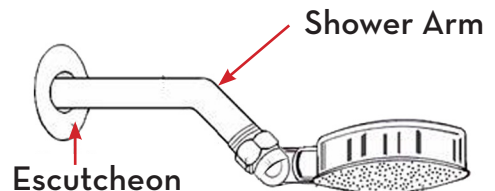
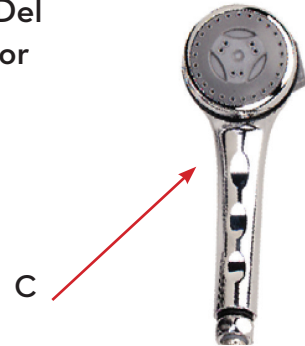
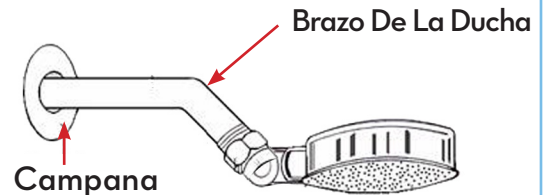
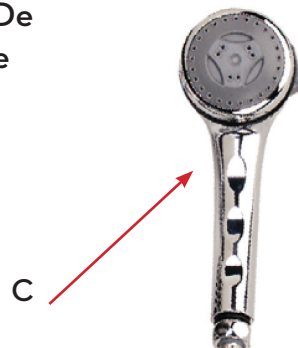
Anti-Siphon

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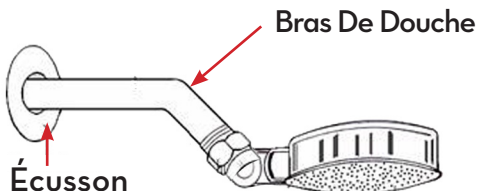## limited lifetime warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

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© 2014 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.