www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

teak bath bench

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banc de bain en teck

EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

assembly instructions

1. Align first support frame with concave side facing up. While holding support frame in position, place second support frame (convex) over first support frame (see figure 1).
2. Be sure screw holes are lined up. Place first Wing Nut Screw in middle (figure 2) to secure both support frames.
3. Place remaining wing nut screws through frame and into bench. Be sure all wing nut screws are tightened securely.
4. Slide adjustable legs over each leg of the support frames. Allow the bronze "push-pins" to "pop out" to lock into position. Adjust to desired height. Be sure all legs are adjusted to the same height.

Contents:	Qty
Teak Bench	1
Support Frame	2
Adjustable Legs	3
Wing Nut Screws	4

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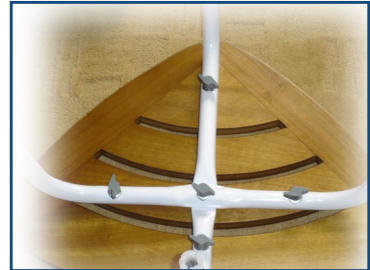
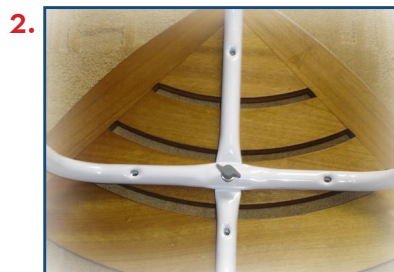
lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2005 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2005 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2005 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.