drive
WWW.DRIVEMEDICAL.COM

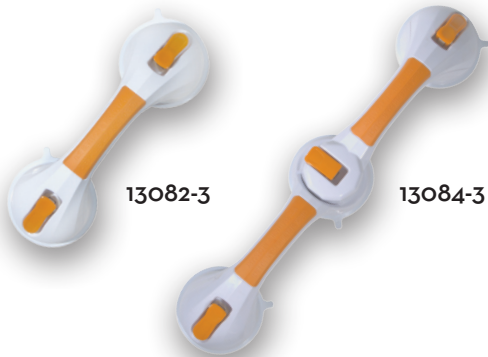
suction cup grab bar
barra de sostén con ventosas
barre d'appui à ventouse

EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

drive
WWW.DRIVEMEDICAL.COM

grab bar

Easily and securely attached without drilling.

Provides support, security, and the convenience of mobility.

Won't rust or corrode.

Drive Medical Suction Grab Bars are convenient, functional and offered in a variety of sizes to fit any lifestyle. They can be attached quickly and just as easily removed! The suction pads are specifically designed to adhere to any smooth, pore-tight surface without drilling! Push down on the vacuum toggle lever and the handles will remain securely in place.

CAUTION

Grab bar is to be used as a balance assist only. Not for body weight leverage. Always make sure the grab bar is securely attached before each use and always reattach if loose.

Always test the holding grip on each surface, each time you apply the suction pad to prevent accidents.

installation instructions

1. Check the vacuum suction discs and the surface it is to be attached to.
2. Clean the surface thoroughly.
3. Press the rubber discs firmly to the surface. The tilting vacuum levers must be loose at this point and the whole of each rubber disc must lie on the surface.
4. Ensure that the rubber discs are completely on the tile area or other surfaces so that the discs do not cover any grouting lines or other joints. The discs should be positioned at least 1/2" away from all joints.
5. First press down one tilting lever and then the second. The resistance of the negative pressure generated must be clearly felt. The greater the resistance felt on the vacuum lever, the better the suction power of the grab bar. The indicator on the side of the suction cup will change from RED to GREEN when a safe and secure hold is achieved. If you use the bar regularly, you will get a feel for the vacuum strength and the suction power generated.
6. Before each use, test the firm hold of the suction device making sure both indicators are GREEN. If the adhesion seems insufficient, release the rail and reattach again. If the required suction power has not been achieved, the rail should not be used.
7. Item # 13084-3 can easily be adjusted by releasing the middle suction cup and one end suction cup and rotating to the desired position. Once the desired position is achieved (making sure the suction cups do not cover any grouting lines or joints), repeat Step 5 for both suction cups.

removal instructions

1. Release both latches.
2. Slide fingertip under rubber seal at each end to release suction.

INCORRECT

CORRECT

maintenance

1. Always keep the rubber discs clear of oil and dust.
2. Clean the rubber discs with standard household cleaning agents.
3. Never expose the rubber discs to sunlight for long periods otherwise they will quickly lose their elasticity.
4. Check the rubber discs for possible deterioration. If the suction cups show damage, replace immediately.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

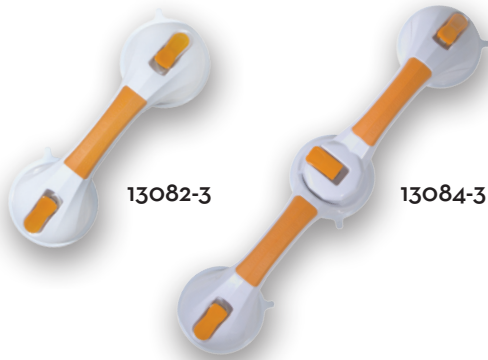
In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.